TCL 300

**Bedienungsanleitung
Operating Instructions
Mode d'emploi
Bedieningshandleiding
Istruzioni d'uso
Instrucciones de uso**

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- 1 ALARM STOP button
- 2 Indicator "Alarm set"
- 3 Volume setting
- 4 Alarm mode selection
(Buzzer + Shaker, Shaker, Buzzer, Off)
- 5 Buttons for setting clock / alarm
- 6 Snooze function setting
- 7 Alarm tone setting
- 8 Battery compartment (for back-up battery)
- 9 Vibration function setting
- 10 Shaker jack
- 11 Power supply jack

Box contents

- Alarm clock
- Power supply 12 V, 800 mA
- Shaker
- User guide

Helpline UK 0844 800 6503

(See chapter "Helpline" for costs and hours of operation)

Installation

Attention: Only use the power adapter plug contained in the material supplied since other power adapter plugs could damage the product. Ensure access to the power adapter plug is not obstructed by furniture or such.

To avoid possible danger from the shaker cord, ensure that the cord is pulled back under the mattress/pillow, and is not visible or reachable.

Back-up battery

With a back-up battery installed, the clock will keep the time and alarm settings in the event of a power cut or the DC adaptor being unplugged.

During a power cut, the clock will not display the time, but the AUDIO will function. If SHAKER is selected, only the ALARM STOP button will light.

If no battery is installed and a power cut occurs, then the clock will lose its time and alarm setting. This is important as if there is a power cut in the night then the alarm will not be set correctly to wake you up in the morning at the required time.

To open the battery compartment please use a suitable implement such as the tip of a ball point pen or small screwdriver.

When connecting the battery observe the correct polarity.

Check the battery condition after one year, replace an old battery. A leaking battery can destroy your clock.

Setting the time

1. Press and hold down the **SET CLOCK** button.
2. Press **HOUR** to set the hours and press **MIN** to set the minutes.
3. Release the **SET CLOCK** button.

Setting the alarm time

1. Press and hold down the **SET ALARM** button.
2. Press **HOUR** to set the hours and press **MIN** to set the minutes.
3. Release the **SET ALARM** button.

Setting the alarm mode

1. Slide the alarm mode switch **MODE** to **OFF** to switch off the alarm function or to stop the current alarm.
2. Slide the alarm mode switch **MODE** to **AUDIO** (Buzzer), **SHAKER** (Vibration) or **ALL** (Buzzer +

Vibration) to select the alarm mode and to switch the alarm function on.

Note: When set to **ALL**, **AUDIO** and **SHAKER** the indicator “Alarm set” is illuminated.

Setting the vibration mode

1. Select the mode with the **VIBRATION** switch:
 - **PULSE**: with interruption
 - **CONTINUE**: continuous.

Setting the snooze mode

1. Select the mode with the **AUTO SNOOZE** switch:
 - **OFF**: snooze function off
 - **ON**: After pressing the **ALARM STOP** button the alarm will be repeated after approx. 9 minutes (total time of activation is approx. 2 hours).

Setting the volume

1. Adjust the alarm volume with **VOL**.

Setting the alarm tone

1. Adjust the alarm tone with **TONE**.

Checking your alarm setting

1. Slide the **AUTO SNOOZE** switch to position TEST.

The selected alarm setting (MODE) goes off.

Note: in TEST mode the LED light (SNOOZE button) does not flash.

Specifications

Functions: Clock, Alarm clock

Alarm modes: Buzzer, Vibration, Buzzer & Vibration

Operating voltage: Power supply Input 100~240 V AC,
Output 12 V DC / 800 mA.

Back-up battery (optional): 9 V battery, PP3 / MN1604,
change once a year

Safety Information

Please read this operating instruction manual thoroughly.

Hazards due to electricity

- Protect the devices from dripping or spraying water. Do not place any containers with liquid, such as vases, on or near the devices. The container may tip over and the liquid may limit the electrical safety.
- Do not operate the devices and power supplies in the open-air or in very humid areas
- Only connect the device to a properly installed socket with a mains voltage that corresponds to the voltage on the power supply.
- Unplug the power supply from the socket:
 - during storms
 - before you clean the connected device

- if faults arise during operation
- if you do not use the device for a longer period of time.

Always unplug the power supply, not the connection cable.

- Do not open or remove parts of the casing under any circumstances. Inside the casing are live parts that can cause electric shock if touched.
- Do not insert any objects into the openings on the device. They might come into contact with live parts.
- The connection cables must not be bent or crushed. Keep the connection cable away from hot surfaces and sharp edges.
- Check the devices and the power supply for damage regularly. Do not turn on any device if one of its components, the power supply or the connection cable has visible damage or a device has fallen.

Disposal

You are obliged to dispose of consumable goods properly in accordance with the applicable legal regulations. The symbol on this product indicates that electrical and electronic apparatus must be disposed of separately from domestic waste at suitable collection points provided by the public waste authorities. Packaging materials must be disposed of according to local regulations.

GB

Maintenance

- Please clean your equipment's surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

Guarantee

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life.

The period of guarantee is 24 months from the date of purchase. The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products.

All deficiencies resulting from material of production faults which occur during the period of guarantee will

be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property. Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOMMS equipment together with the purchase receipt. All rights to

claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

Helpline

If you need some help and you're in the UK, please call us on 0844 800 6503 (using an UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays.

If you've a claim on your 2-year guarantee, please contact your supplier.

Declaration of conformity

This device fulfils the
requirements stipulated
in the EU directive:

2004/108/EC relating to electromagnetic compatibility.

Conformity with the above mentioned directive is
confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity,
please refer to the free download available on our
website **www.amplicomms.com**.

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Audioline GmbH