# SA-40

## Cell Phone Amplifier

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**User Guide**

**Mode d'emploi**

**Istruzioni d'uso**

**Handleiding**

**Manual de  
instrucciones**

**Übersicht  
At a glance  
Aperçu  
Riepilogo  
Overzicht  
Información general**

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## At a glance

- |   |                    |    |                         |
|---|--------------------|----|-------------------------|
| 1 | Mute button        | 9  | Mode switch             |
| 2 | Mute indicator LED | 10 | Clip Holder slot        |
| 3 | Talk button        | 11 | Volume                  |
| 4 | Power on LED       | 12 | Tone                    |
| 5 | Loudspeaker        | 13 | Micro USB charging jack |
| 6 | Battery LED        | 14 | Battery                 |
| 7 | Battery cover      | 15 | Headset jack            |
| 8 | On-/Off-switch     |    |                         |

## Scope of delivery

- |   |          |   |                  |
|---|----------|---|------------------|
| 1 | SA-40    | 1 | AC power adapter |
| 1 | Batterie | 1 | User guide       |
| 1 | Clip     |   |                  |

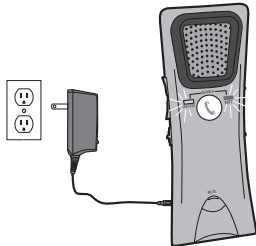
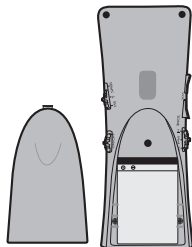
## Install battery

- 1 Install battery as shown.
2. Charge before use. Allow approximately 3-4 hours for the charging process.

## Charging

- 1 When your SA-40's battery charge is low, the Battery Indicator will will continuously flash.
2. To charge the SA-40, connect AC Adapter as shown. Typical charging to full charge is 3-4 hours. While the SA-40 is charging, the Battery Indicator Light will be steadily lit.
4. When the the SA-40 is fully charged, the Battery Indicator Light will go out.
5. The SA-40 should never be connected to the AC Power Adapter for more than one week.

## Presets

- Mode of Operation: Handset
- Volume: Medium
- Tone: Medium

## Pairing the SA-40 with your cell phone

- 1 Activate the Bluetooth® feature of your cell phone.
- 2 Turn on the SA-40. It will blink as it pairs to your cell phone. Follow the prompts on your cell phone to complete pairing. If your cell phone asks for a Pin Number, enter: 0000. The SA-40 will remain paired until you delete it from your cell phone.

## Select an Operating Mode

Slide the mode switch on the right side of your phone to your preferred mode.

- HS: Use the SA-40 as a handset.
- T: Couple with your T-Coil Hearing Aid to hear through your hearing aid.
- SPKR: Use as a speakerphone, for hands-free conversation.

### **Please note:**

- **The SA-40 amplifies your caller's voice up to 100 times louder than your cell phone. Use care when setting phone volume to avoid damage to your hearing.**
- **The SA-40 should always be set to (HS) mode when handset is pressed to the ear.**  
**Do not press the SA-40 to your ear if in SPKR Mode.**
- **Some cellphones may produce more noise than others. This noise is also a function of the strength of the cell phone signal, which can vary from place to place.**

## **Answer a call**

- 1 When your phone rings, press  to answer.
- 2 Adjust volume to the desired level by pressing the VOL(+) and (-) buttons. You will hear a beep when the volume has reached the minimum and maximum volumes.
- 3 To enhance clarity, adjust the tone by moving the tone switch to the setting where you hear best.
- 4 Press  to end the call.

## **Making a call**

- 1 Press .
- 2 If your cell phone is equipped with voicemail, speak the desired name or number into the SA-40.

- 3 If your cell phone does not have voicemail, enter the desired number on your cell phone keypad.
- 4 Press  to end the call.

## **Using your SA-40 in your car**

- 1 Attach the visor clip securely to the car's visor.
- 2 Set the SA-40 to SPKR (Speakerphone) mode and attach it to the visor clip (provided). Always follow all local, city and state driving regulations regarding cell phone calls while driving.

## **Using your SA-40 with your T-Coil hearing aid:**

- 1 Set the mode selector to "T" (T-Coil mode) not SPKR mode. No sound will be emitted from the SA-40 speaker.
- 2 Place speaker grill of the SA-40 over your hearing aid.

## Using the Mute button

- 1 Press the mute button to mute the microphone on the SA-40. The mute indicator will light up. Press it again to un-mute.

## Troubleshooting

- Why can't my caller hear me?  
You may have your mute button on.
- My SA-40 is turned on and is paired with my cell phone. Why can't I receive or make calls?  
You may be out of range. Your SA-40 works within 30 feet from your paired cellphone.

## Safety Instructions

- 1 Use only the supplied rechargeable battery or one of comparable type. Do not use regular batteries!
- 2 Only use accessories approved by the manufacturer.
- 3 Do not use the device in direct proximity to water.
- 4 When unplugging connections, never pull on the cable. Always handle by the plug/casing. Disconnect the charger from the power if it is not use or when you want to clean it. Never use a device with a damaged connection cable or casing or if it has fallen or been otherwise damaged. Bring the device to an authorized repair shop to have it examined for damages or repaired.
- 5 While driving a car, your complete attention must be on the road.

Follow all applicable regulations for talking on the phone while driving a car.

- 6 Do not use the device during a storm.
- 7 Do not use the device to report a gas leak near the leak.
- 8 Pay special attention to handling the device when you allow children to use it.

## Power adapter plug

**Attention: Only use the power adapter plug contained in the material supplied since other power adapter plugs could damage the transmitter. Ensure access to the power adapter plug is not obstructed by furniture or such.**

Pay attention that the plug and cable are in perfect condition. Kinked or worn cable represents the risk of a fatal accident!

The power adapter plug supplied fulfils the ecodesign requirements of the European Union (Directive 2005/32/EC). This means that, both in an operating state and in an idling state, the power consumption is considerably lower compared to power adapter plugs with an older design.

- Energiesparend
- Economisant l'énergie
- A risparmio di energia
- Energy efficient

## Disposal

In order to dispose of your device, take it to a collection point provided by your local public waste authorities (e.g. recycling centre). According to laws on the disposal of

electronic and electrical devices, owners are obliged to dispose of old **electronic and electrical devices** in a separate waste container. The adjacent symbol indicates that the device must not be disposed of in normal domestic waste!

**Batteries represent a hazard to health and the environment!**

Never open, damage or swallow batteries or allow them to pollute the environment. They may contain toxic, ecologically hazardous heavy metals. You are legally obliged to dispose of power packs and batteries at the point of sale or in the corresponding containers provided at collection points provided by local public waste authorities. Disposal is free of charge. The adjacent symbols indicate that the

batteries must not be disposed of in normal domestic waste and must be brought to collection points provided by local public waste authorities.

**Packaging materials** must be disposed of according to local regulations.

## Maintenance

- Clean the housing surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

## Technical data

Talk Time	Up to 10 hours
Standby Time	Up to 14 days

Operating Range	Up to 33 feet from phone (with optimal conditions)
Charge Connector	Micro USB
Battery	3.7 V / 800 mAh Lithium Ion
Charge Time (Max)	Approx. 3-4 hours to full charge
Charger	Ausgang: 5 VDC, 400 mA
Bluetooth® Version	v.2.1 + EDR
Bluetooth® Profiles	HFP, HSP, A2DP
Operating temperature	0° C - 40° C
Storage temperature	-10° C - 50° C

## Helpline

If you need some help and you're in the UK, please call us on 0844 800 6503 (using a UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays. If you've a claim on your 2-year guarantee, please contact your supplier.

## Declaration of conformity

This device fulfils the requirements stipulated in the EU directive:

1999/5 EU directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity.

Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website **[www.amplicomms.com](http://www.amplicomms.com)**

## **Guarantee**

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life.

The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material or production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer. If your equipment shows signs of defect during the period of guaran-

tee, please return to the sales outlet in which you purchased the AM-PLICOMMS equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

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Audioline GmbH, D-41460 Neuss