# TV 2510NL

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## Notizen

## **Receiver**

- 1 Microphone button for room monitoring
- 2 3.5 mm Audio output jack
- 3 Microphone
- 4 Volume control
- 5 Power on indicator
- 6 On/Off switch
- 7 Volume balance (left/right)
- 8 Battery

## **Transmitter base**

- 1 Charging cradle
- 2 LED indicator of tone clarity
- 3 Tone clarity button
- 4 LED indicator of power
- 5 LED charging indicator for main battery (attached to neck-loop receiver)
- 6 LED charging indicator for spare battery
- 7 Spare charging slot
- 8 DC In
- 9 Analogue Audio In Socket (3.5 mm)
- 10 Optical Audio In Socket TOSLINK
- 11 Mono/Stereo Switch

**Please note that only one input (either analogue audio input or optical input/TOSLINK) must be connected.**

# Getting started

## Getting started

### Box contents

- 1 Receiver with induction neck-loop
- 1 Transmitter base
- 1 AC adapter for the transmitter
- 2 Rechargeable batteries for the receiver (Lithium Polymer)
- 1 Optical TOSLINK Digital Audio cable
- 1 Analogue Audio cable (both ends with 3.5 mm stereo plug)
- 1 Adapter cable (RCA) for analogue audio cable
- 1 Adapter plug 6.3 mm to 3.5 mm for analogue audio cable
- 1 External microphone with holder and adapter plug for analogue audio cable
- 1 User guide

**Important: please make sure you've read the safety information on page 30 before you set up your device.**

### Connecting the transmitter

- 1 Connect the power adapter to the DC IN input socket located on the transmitter rear panel.
- 2 Connect the power adapter provided to an electrical outlet.
- 3 Connect the transmitter to your audio source.  
RCA adapter cable, audio cable, 6.3 mm to 3.5 mm adapter plug and an external microphone are provided for various type of connection to your audio source (e.g. TV set, Hi-Fi system, MP3 or iPod).

**Please note that only one input (either analogue audio input or optical input/TOSLINK) must be connected.**

The green power indicator LED will blink as soon as you have connected the power adapter to the electrical outlet and when the audio signal is detected from the audio source, the green LED indicator will glow constantly.


## Using the optical TOSLINK digital audio cable

If your audio source has an optical audio output (TOSLINK/S-PDIF), connect the two devices with the included optical TOSLINK digital audio cable. This connection offers a higher transmission quality and a lower susceptibility to electrical and magnetic interference.

**Please make sure the optical TOSLINK digital audio cable does not get bent or folded! Avoid tight bending radii.**


## Using the adapter (only for analog audio cables)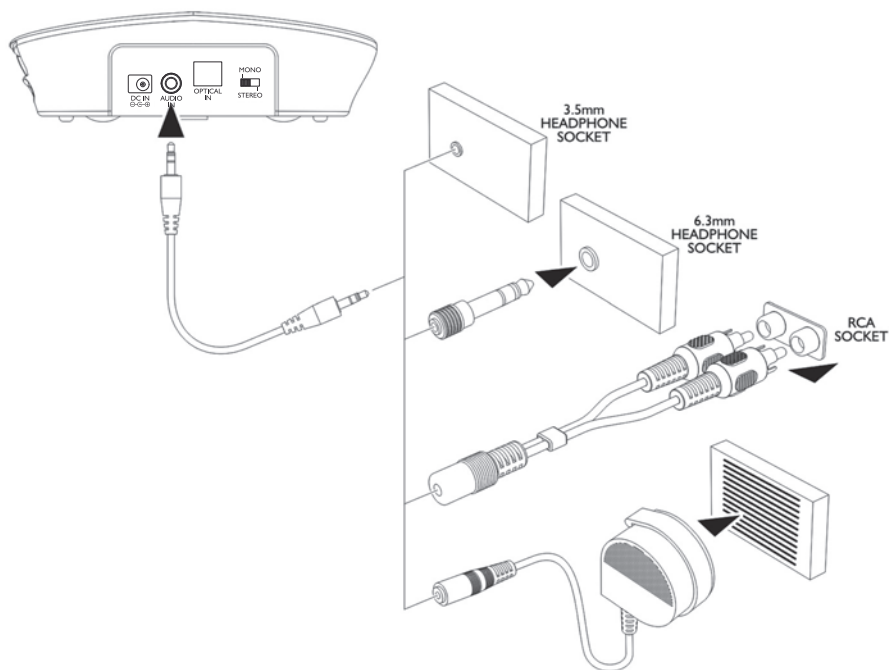

## Set up the receiver

### Installing and charging the battery


1 Insert a battery into the receiver with correct polarity.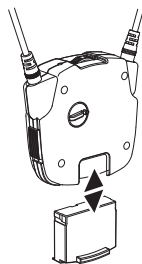


2 Charge the battery for about 3 hours before first use.

Switch off the receiver (position OFF).

Place the receiver into the charging cradle of the transmitter. The charging process starts automatically. The battery LED indicator will be red and will turn to green when the battery is fully charged. This takes about 3 hours.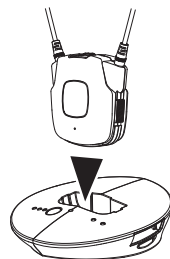

3 Insert the spare battery into the base as indicated in the drawing. The LED will remain red and turn green when it is fully charged after about 2 hours and 30 minutes.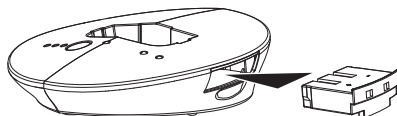

- A fully charged battery provides up to 5 hours of listening enjoyment.
- To conserve battery power, the receiver automatically shuts off if no signal is detected for a period of approx. 5 minutes or when the level is too low. However, if the signal returns to an acceptable level, the transmitter will automatically switch ON again.


## Additional earphone/headset jack

You may connect a usual earphone/headset (3.5 mm) to the socket placed on the side of the receiver.

Please note, that the transmission via the neck-loop induction cable is switched off when an earphone/headset is connected.

## External microphone

If your TV set or HiFi system does not have any audio jack you should use the external microphone.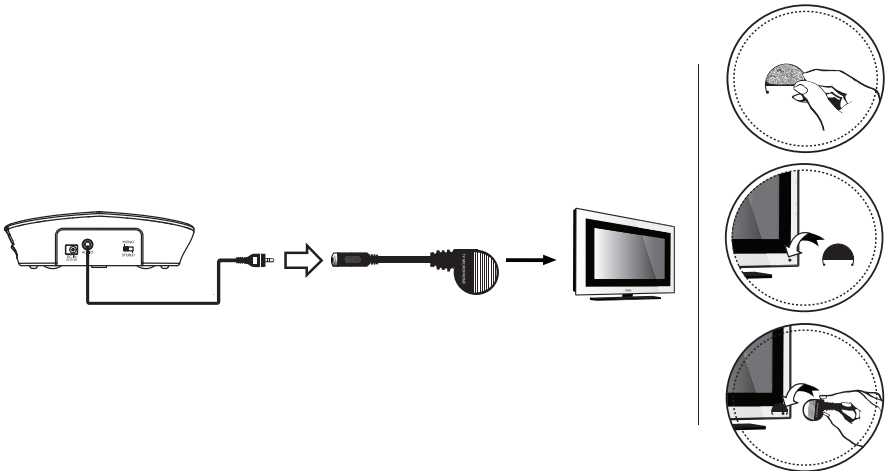

- 1 Connect the analogue audio cable to the microphone.
- 2 Fasten the microphone holder using the adhesive strip adjacent to the speaker grill on the TV and then clip the microphone into the holder so that it locates just above the speaker.
- 3 Please initially set a low volume on the TV/Hi Fi system and adjust to the required volume in conjunction with the volume control on the headset. The position of the microphone has a decisive influence on the volume - please determine the best position by experimentation.

**Note:** The output will be in mono when using the microphone.

**Please make sure that the microphone is not too close to the transmitter, otherwise this will result in interference being picked up by the microphone which will be heard as digital noise in the receiver.**

## Setting up an additional receiver (TV 2400-1 / TV 2410NL-1)

**Your TV 2400-1 / TV 2410NL-1 is an additional receiver for your TV 2500 / TV 2510NL. After having inserted and charged the battery you will have to register it to the transmitter. There is no limit number of receivers to be registered to the TV 2500 / TV 2510NL transmitter for use together.**

### Inserting and charging the rechargeable battery

- 1 Insert a battery into the headset receiver.
- 2 Charge the battery before first use. Switch off the receiver (position OFF). Place the receiver into the charging cradle of the transmitter. The charging process starts automatically. The battery LED indicator will be red and will turn to green when the battery is fully charged. This takes about 2 hours and 30 minutes.  
Alternatively you can use the fully charged spare battery from the transmitter.

### Registering the receiver

- 1 Turn off the transmitter if it is on (disconnect from the mains).
- 2 Disconnect the audio source input.
- 3 Turn on the NEW headset, the LED indicator will be flashing on and off slowly.
- 4 Press the MIC button and hold for 3s, the green LED indicator will flash on and off quickly.
- 5 Turn on the transmitter immediately (must be done within 9s, or need to redo step 4).
- 6 If the registration is successful, you will hear a short confirmation tone in the headset and the green light on the headset will automatically stop flashing and will light constantly. This indicates that registration was successful.
- 7 Reconnect the audio source input.

# Operation

## Operation

### Hearing damage due to high volumes!

- Before putting the receiver on lower the volume setting and gradually increase the volume to suit.
- Do not continuously expose yourself to high volumes.

- 1 Switch on the audio source. The green indicator LED on the transmitter will glow constantly.
- 2 Switch the On/Off switch on the receiver to ON position,
- 3 The neck-loop induction cable can transmit the audio signal to the hearing aid device for listening (**Note** : any earphone/headset must be pulled out from the 3.5 mm audio output jack).
- 4 Adjust the tone clarity setting to achieve optimum performance.

**Note:** When the battery level is low you will hear 3 beeps every minute in the receiver. The receiver will be switched off automatically after 5 minutes.

### The microphone button

The microphone opening at the side of the receiver pick up the sounds of your surroundings. When the microphone is switched on, you will hear your surroundings louder. You can hear for example your door bell, your telephone or your conversational partner louder when you are wearing the receiver.


Press the button to switch the microphone on. The power on indicator will turn from green to red and two short beeps are to be heard. Make sure that the microphone opening is not covered!

You can hear your surroundings and you use the receiver as an assistive listening device which helps you hear other people better.



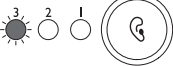
To switch the microphone off press the button again. The power on indicator will turn to green and you will hear the sound from the audio source.

## Tone clarity

You can set the tone clarity with the key on the transmitter. Press and release the required key repeatedly. The current setting will be indicated by the status of the three LED's.

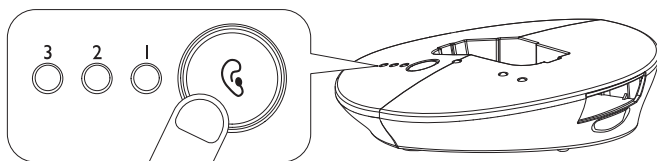The following table shows the settings. Try out which setting works best for you.

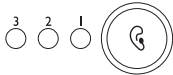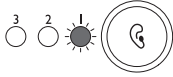Setting	LED indicator (orange)	Effect
Default		Flat
1		Bass Boost + Treble Boost
2		Treble Boost
3		Bass Boost

## Stereo / mono switch - transmitter base

Although the output from the neck loop will always be a mono signal, if the source output is in stereo, the resultant output can be affected as follows:-

If you want both stereo channels from the source signal to be equally blended into a mono signal, switch to mono (this is the recommended setting).

Left or right channels can be partially filtered by switching to stereo and adjusting the balance control on the headset.

## **Troubleshooting**

### **No sound**

- Ensure that the mains adapter is pushed fully into the power outlet and that its cable is properly connected to the DC IN connector on the transmitter.
- Ensure that the receiver power ON/OFF switch is switched to ON.
- The charge level of the battery inside receiver may be too low. Recharge the battery or replace it with a fully charged battery.
- Ensure that the TV set, Hi-Fi system or audio component is switched ON and set to a channel that broadcasts an audio signal.
- Use the volume control on the receiver to increase the sound level.
- The connected audio/video equipment may not be in playback mode. Start playback on by the equipment.
- The receiver output level is set too low. Adjust the volume to a suitable level.
- Try using a different connection RCA or HEADPHONE or use the supplied microphone.

### **Distortion**

- The charge level of the battery inside the receiver may be too low. Recharge the battery or replace them with a fully charged battery.
- The receiver may be too far away from the transmitter. Move closer to it.
- The audio signal input level is too low. Increase the volume from the audio source.
- The audio signal input level may be too high, lower the volume from the audio source.

# Appendix

## Appendix

### Safety Information

- Please read this operating instruction manual thoroughly.
- **For safety reasons do not use this product near machinery as neckloop can become tangled in moving parts.**

### Intended use

This device was developed to transmit audio signals from an audio source like TV or HiFi device to a receiver, using cordless transmission. Any other use is considered unintended use.

Unauthorised modification or reconstruction is not permitted. Under no circumstances open the devices or complete any repair work yourself.

### Installation location and safety information

- Prevent excessive exposure to smoke, dust, vibration, chemicals, moisture, heat and direct sunlight.
- Furniture polish, over time, may disintegrate the rubber feet of the transmitter which may stain your furniture. To avoid a potential stain, you may want to place the transmitter on a pad or mat.
- This listener allows you to amplify sounds to a loud volume, but listening to earphones at a high volume over an extended period of time may damage your hearing; especially in children. Thus, we suggest that you set the volume to a level that is comfortable to your hearing.

### For users wearing a pacemaker

It is recommended that if you have a pacemaker fitted you check with a medical expert or your health adviser before using this product. Switch the device off immediately if you notice or suspect any adverse effects.

## Power adapter plug

**Attention: Only use the power adapter plug contained in the material supplied since other power adapter plugs could damage the transmitter. Ensure access to the power adapter plug is not obstructed by furniture or such.**

Pay attention that the plug and cable are in perfect condition. Kinked or worn cable represents the risk of a fatal accident!

The power adapter plug supplied fulfils the ecodesign requirements of the European Union (Directive 2005/32/EC). This means that, both in an operating state and in an idling state, the power consumption is considerably lower compared to power adapter plugs with an older design.

- Energiesparend
- Economisant l'énergie
- A risparmio di energia
- Energy efficient

## Rechargeable batteries

**Attention: Never throw batteries into a fire. Only use batteries of the same type! Pay attention to correct polarity.**

## Disposal

In order to dispose of your device, take it to a collection point provided by your local public waste authorities (e.g. recycling centre). According to laws on the disposal of electronic and electrical devices, owners are obliged to dispose of old **electronic and electrical devices** in a separate waste container. The adjacent symbol indicates that the device must not be disposed of in normal domestic waste!

### **Batteries represent a hazard to health and the environment!**

Never open, damage or swallow batteries or allow them to pollute the environment. They may contain toxic, ecologically hazardous heavy metals. You are legally obliged to dispose of power packs and batteries at the point of sale or in the corresponding containers provided at collection points provided by local public waste authorities. Disposal is free of charge. The adjacent symbols indicate that the batteries must not be disposed of in normal domestic waste and must be brought to collection points provided by local public waste authorities.

**Packaging materials** must be disposed of according to local regulations.

### **Helpline**

If you need some help and you're in the UK, please first check and see if your problem can be solved using the troubleshooting guide on page 28. If it can't, please call us on 0844 800 6503 (using a UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays.

If you've a claim on your 2-year guarantee, please contact your supplier.

### **Declaration of conformity**

This device fulfils the requirements stipulated in the EU directive: 1999/5 EU directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity.

Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website **[www.amplicomms.com](http://www.amplicomms.com)**

## Maintenance

- Clean the housing surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

## Technical data

Transmission mode:	Digital hopping
Carrier frequency:	2.4GHz ISM Band
Range:	Up to 25 meters (open area)
Frequency response:	30 Hz – 20 KHz
Distortion:	< 0.5%
Signal-to-noise ratio:	> 75 dB
Operating voltage (Transmitter):	5 VDC, 550 mA
Operating voltage (Receiver):	3.7 V, 350 mA Lithium Polymer Battery
Operation time:	With fully charged battery up to 5 hours
Charging time:	For empty battery approx. 3 hours
Weight Receiver	78 g
Weight Transmitter	115 g

## Guarantee

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life.

The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material or production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOMMS equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

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Audioline GmbH, D-41460 Neuss