TV 200

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User Guide

Mode d'emploi

Istruzioni d'uso

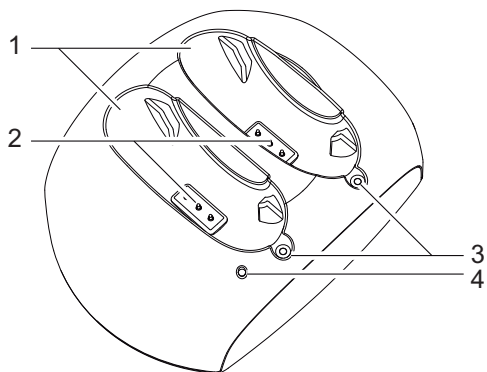
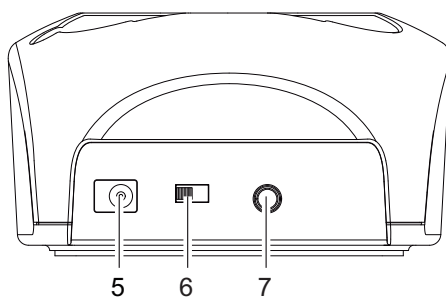
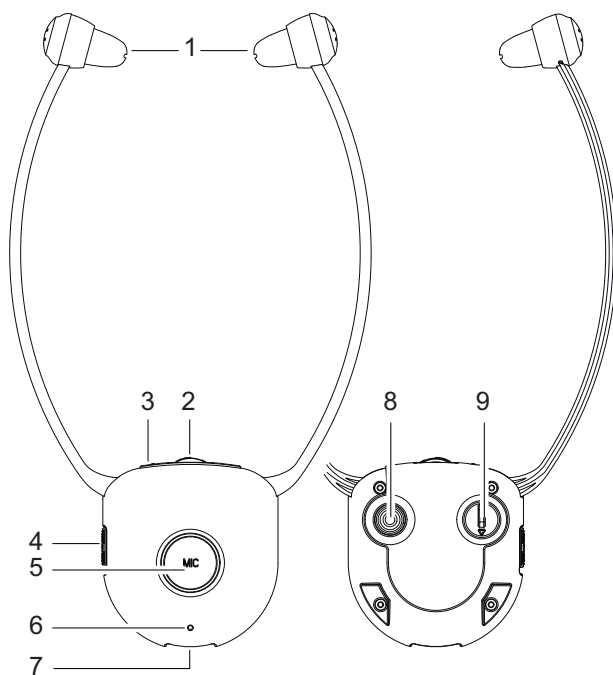
Handleiding

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Bruksanvisning

**Manual de
instrucciones**

Empfänger

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Receiver

- 1 Earbuds
- 2 Volume control
- 3 Power/Signal LED indicator
- 4 Power On/Off switch
- 5 Mute button for verbal communications (via microphone)
- 6 Microphone
- 7 Charging contacts
- 8 Tone adjustment
- 9 Sound balance control

Transmitter

- 1 Charging cradle
- 2 Charging contacts
- 3 LED indicator of charging (Red: Charging, Green: Fully charged)
- 4 LED indicator of Power/Audio signal
- 5 DC in
- 6 Power ON/OFF
- 7 Audio In (3,5 mm)

Box contents

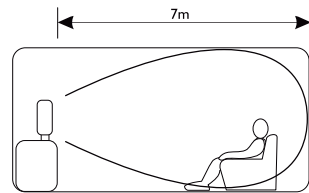
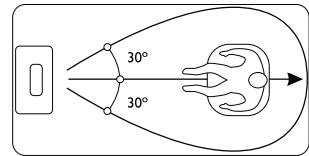
- 1 Receiver
- 1 Transmitter base
- 1 Additional pair of rubber ear buds
- 1 Audio cable (both ends with 3.5 mm stereo plug)
- 1 Adapter plug 3,5 / 6,3 mm
- 1 Adapter cable (RCA)
- 1 AC adapter for the transmitter
- 1 User guide

Getting started

Important: please make sure you've read the safety information on page 15 before you set up your device.


Positioning of the transmitter

Place the transmitter unit so that the IR Transmitter LED approximately faces the center of your listening area. Make sure, that there are no obstacles between the transmitter and the receiver, which would interfere with the transmission. Transmission range is approximately 7 meters.

Note: Direct sunlight or very bright interior lights can interfere with the transmission. Check in these cases another location or take appropriate measures to shadowing.

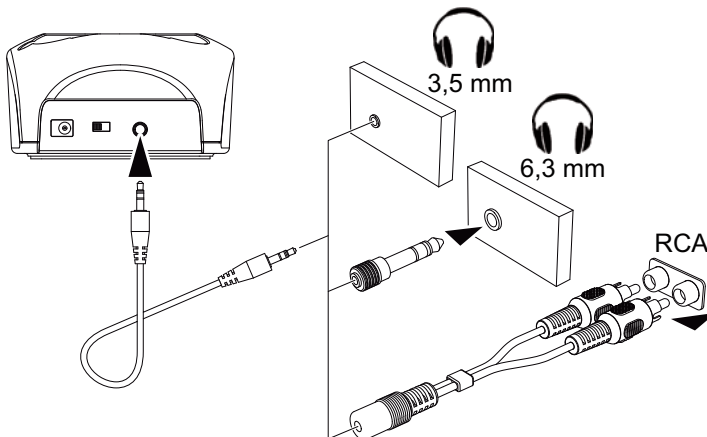
Connecting the transmitter

- 1 Connect the power adapter to the DC IN input socket located on the transmitter rear panel and connect the power adapter to an electrical outlet.
- 2 Connect the transmitter with the audio source using the audio cable provided. If necessary, use one of the audio cable adapters.
- 3 Slide the power-on switch on the rear side of the transmitter to the position **ON**. The power indicator LED will light green.

Set up the receiver

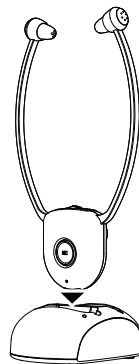The receiver comes with a built-in rechargeable battery. Place the receiver into the charging cradle and make sure the battery is fully charged before first use (charging LED indicator lights red). This will take approx. 2 hours. If the battery is fully charged, the charging LED indicator lights green. A fully charged battery provides up to 9 hours of listening enjoyment.

- 1 Take the fully charged receiver from the cradle and switch it on (ON). The power indicator LED will light green.
- 2 Switch on the audio source.
- 3 Flex the head set arms outwards until the ear buds can be inserted in the ear. Please note the ear buds can be angled through a movement up, down or sideways, so as to enter the ear a little deeper. This will improve the sound quality and volume. Gently press the ear buds further into the ear to find a comfortable position for the ear buds and an ideal position for the sound.

Hearing damage due to high volumes!

This receiver allows you to set higher volumes than conventional headphones. This can cause hearing damage in persons with healthy hearing and especially in children.

- **Before putting the receiver on, set the volume on the receiver to a low value.**
- **Do not continuously expose yourself to high volumes.**

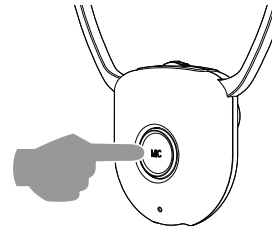
Note:

- The LED indicator on the receiver will blink slowly red with “Beep” sound in every minute when the battery level is low. Recharge the battery. Always ensure that the Headphone Receiver is switched to OFF position while the batteries are charging.

Communication via the built-in microphone

The microphone opening at the front of the receiver pick up the sounds of your surroundings. When the microphone is switched on, you will hear your surroundings louder. You can hear for example your door bell, your telephone or your conversational partner even when you are wearing the receiver.

Press the button to switch the microphone on. The power on indicator will turn from green to red. Make sure that the microphone opening is not covered! You can hear your surroundings and you use the receiver as an assistive listening device which helps you to better hear other people.

To switch the microphone off press the button again. The power on indicator will turn to green and you will hear the sound from the audio source.

Tone adjustment

When necessary, treble or bass can be adjusted by the tone clarity button on the receiver. Simply press the tone button to select the desired needs:

- 1 short beep - Treble Boost
- 2 short beeps - Bass Boost
- long beep - Flat

Changing the rubber ear buds

Two pairs of ear buds are delivered with the device (L=large/S=small).

The ear buds "S" (small) are already mounted on the headset.

- 1 For changing the ear buds pick the mounted ear buds at the end and tear them off simply.
- 2 Position the spare ear buds onto the "open ends" and let them snap into place.

Additional receivers

You can use any number of receivers (headsets) with one transmitter. An additional receiver (headset) is available at your dealer.

Your TV 200-transmitter can charge the rechargeable batteries of two receivers (headsets) simultaneously.

Appendix

Safety Information

Please read this operating instruction manual thoroughly.

Intended use

This device was developed to transmit audio signals from an audio source like TV or HiFi device to a receiver, using IR transmission. Any other use is considered unintended use.

Unauthorised modification or reconstruction is not permitted. Under no circumstances open the devices or complete any repair work yourself.

Installation location and safety information

- Prevent excessive exposure to smoke, dust, vibration, chemicals, moisture, heat and direct sunlight.
- Furniture polish, over time, may disintegrate the rubber feet of the transmitter which may stain your furniture. To avoid a potential stain, you may want to place the transmitter on a pad or mat.
- This listener allows you to amplify sounds to a loud volume, but listening to earphones at a high volume over an extended period of time may damage your hearing; especially in children. Thus, we suggest that you set the volume to a level that is comfortable to your hearing.

Power adapter plug

Attention: Only use the power adapter plug contained in the material supplied since other power adapter plugs could damage the transmitter. Ensure access to the power adapter plug is not obstructed by furniture or such.

Pay attention that the plug and cable are in perfect condition. Kinked or worn cable represents the risk of a fatal accident!

The power adapter plug supplied fulfils the ecodesign requirements of the European Union (Directive 2005/32/EC). This means that, both in an operating state and in an idling state, the power consumption is considerably lower compared to power adapter plugs with an older design.

- Energiesparend
- Economisant l'énergie
- A risparmio di energia
- Energy efficient

Disposal

In order to dispose of your device, take it to a collection point provided by your local public waste authorities (e.g. recycling centre). According to laws on the disposal of electronic and electrical devices, owners are obliged to dispose of old **electronic and electrical devices** in a separate waste container. The adjacent symbol indicates that the device must not be disposed of in normal domestic waste!

Batteries represent a hazard to health and the environment!

Never open, damage or swallow batteries or allow them to pollute the environment. They may contain toxic, ecologically hazardous heavy metals. You are legally obliged to dispose of power packs and batteries at the point of sale or in the corres-

ponding containers provided at collection points provided by local public waste authorities. Disposal is free of charge. The adjacent symbols indicate that the batteries must not be disposed of in normal domestic waste and must be brought to collection points provided by local public waste authorities.

Packaging materials must be disposed of according to local regulations.

Helpline

If you need some help and you're in the UK, please call us on 0844 800 6503 (using a UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays.

If you've a claim on your 2-year guarantee, please contact your supplier.

Declaration of conformity

This device fulfils the requirements stipulated in the EU directive: 1999/5 EU directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity.

Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website www.amplicomms.com

Maintenance

- Clean the housing surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

Technical data

System	Infrared (IR)
Modulation	FM
Carrier frequency (left/right)	2.3 / 2.8 MHz
Power supply (transmitter)	Power adapter plug, 12 VDC, 1 A
Power supply (receiver)	3.7 V, 350 mA, Lithium Polymer battery
Receiver battery life	Fully charged battery up to 9 hours
Frequency response	60 Hz - 10 kHz
Distortion	approx. 1%
Signal-to-noise-ratio	> 48 dB
Range	up to 7 m (line of sight in open area)

Accessories / Spare parts

Additional receiver: TV 200-1

Replacement earbud covers (2 pairs / 1 pair each S, L): TV 2500-EBC

These accessories and spare parts are available at your AMPLICOMMS dealer or call the Helpline UK.

Guarantee

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life.

The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material or production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOMMS equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

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Audioline GmbH, D-41460 Neuss