BTH 1400

**Bedienungs-
anleitung (DE)**

User Guide (GB)

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Istruzioni d'uso (IT)

Handleiding (NL)

**Manual de
instrucciones (ES)**

Bruksanvisning (SE)

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Bruksanvisning (NO)

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Übersicht At a glance Aperçu Riepilogo Overzicht Información general Översikt Características gerais Oversikt Yleiskuvaus

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- 1 Volume control
- 2 LED indicator
- 3 Off-hook/On-hook button
- 4 Microphone button for room monitoring
- 5 Balance left/right
- 6 Tone clarity button
- 7 Jack for charging cable (micro USB)
- 8 On/Off switch

Getting started

Getting started

Box contents

- 1 Receiver with rubber ear buds size “L” (large)
- 1 AC adapter for charging the receiver
- 1 Additional pair of rubber ear buds size “S” (small)
- 1 User guide

Important: please make sure you've read the safety information on page 24 before you set up your device.

Charging the battery

Charge the battery for about 3 hours before first use. Switch off the receiver (position OFF).

- 1 Attach the micro USB plug (power adapter) to the charging jack on the receiver.
- 2 Attach the power supply to a wall outlet. The LED indicator will flash green whilst the power supply is connected and the unit is charging.

- **A fully charged battery provides up to 5 hours of listening enjoyment.**
- **To conserve battery power, the receiver automatically shuts off if no signal is detected for a period of approx. 5 minutes or when the level is too low.**
- **Standby time: up to 180 hours (when the receiver is connected to a device but not playing any sound)**
- **Auto off: when the receiver is turned on without any connection to Bluetooth devices, it will be turned off after 5 minutes. To turn on the receiver again, you should first slide the power button to the “OFF” position. Then slide it back to “ON” again.**
- **After Auto off: if the power button is not slid to “OFF” the battery can last up to 1.5 months.**

Bluetooth pairing

- 1 Switch on the receiver.
- 2 Press and hold down the MIC button for approx. 5 seconds. The indicator LED will flash green and blue.
- 3 Now run on the transmitter (e.g. on your mobile phone) the necessary steps. If necessary, refer to the relevant user manual for help. The Bluetooth name of the receiver is "amplicomms".
- 4 If the devices are paired successfully, the indicator LED lights blue constantly.

NFC pairing

If your sending device (e.g. your mobile phone/smartphone) supports NFC (Near Field Communication), attach the receivers area marked with the NFC-Logo () as near as possible to the relevant area on the sender for a few seconds. It will then activate the Bluetooth function and pair with each other automatically.

Changing the rubber ear buds

Two pairs of ear buds are delivered with the device (L=large/S=small).

The ear buds "L" (large) are already mounted on the headset.

- 1 For changing the ear buds pick the mounted ear buds at the end and tear them off simply.
- 2 Position the spare ear buds onto the "open ends" and let them snap into place.

Operation

Operation

Hearing damage due to high volumes!

This receiver allows you to set higher volumes than conventional headphones. This can cause hearing damage in persons with healthy hearing and especially in children.

- Before putting the receiver on, set the volume on the receiver to a low value.
- Do not continuously expose yourself to high volumes.

- 1 Switch the On/Off switch on the receiver to ON position.
- 2 Make sure the Bluetooth connection is established (indicator LED lights blue)
- 3 Flex the head set arms outwards until the ear buds can be inserted in the ear. Please note the ear buds can be angled through a movement up, down or sideways, so as to enter the ear a little deeper. This will improve the sound quality and volume. Gently press the ear buds further into the ear to find a comfortable position for the ear buds and an ideal position for the sound.
- 4 Adjust the desired volume.
- 5 Adjust the tone clarity setting if necessary.
To set the tone clarity briefly press the TONE button repeatedly.
 - 1 short beep: Treble boost
 - 2 short beeps: Bass boost and treble boost
 - 3 short beeps: Bass boost
 - 1 long beep: Flat
- 6 Adjust the balance (left/right) if necessary.

Note: When the battery level is low, the indicator LED sparkles in blue (Bluetooth mode) or in green (MIC mode). You will hear 3 beeps every minute and 1 double-beep in the forth minute. The receiver will switch off automatically in the fifth minute.

The MIC button

The microphone opening at the lower part of the receiver front picks up the sounds of your surroundings. When the microphone is switched on, you will hear your surroundings louder. You can hear for example your door bell, your telephone or your conversational partner even when you are wearing the receiver.

Press the MIC button to switch the microphone on. The power on indicator will turn to green and one short beeps is to be heard. Make sure that the microphone opening is not covered!

You can hear your surroundings and you use the receiver as an assistive listening device which helps you to better hear other people.

Press the MIC button again to switch off the microphone. The power on indicator will return to blue and two short beeps are to be heard.

Caution: When switching between the microphone and music source please be aware that the volume level can change substantially dependant upon signal input levels. As a precaution the volume level should be lowered prior to switching between the microphone and music.

The PHONE button

Do you have your receiver connected to a mobile phone / smartphone, you can answer or end a phone call by briefly pressing the phone button. When an incoming call is received the ringing tone from the mobile phone will be transmitted to the headset which in turn will be heard in the earpieces.

Press and release the 'Phone' button on the BTH1400 and then you can converse with the caller. The microphone in the headset will pick up your voice. To end the call simply press and release the 'Phone' button again.

Troubleshooting

No sound

- Ensure that the receiver power ON/OFF switch is switched to ON.
- The charge level of the battery inside receiver may be too low. Recharge the battery for approx. 3 hours.
- Ensure that the Bluetooth connection is established.
- The receiver output level is set too low. Adjust the volume to a suitable level.

Distortion

- The charge level of the battery inside the receiver may be too low. Recharge the battery for approx. 3 hours.
- The receiver may be too far away from the Bluetooth transmitter (e.g. smartphone etc.). Move closer to it.

Appendix

Appendix

Safety Information

Please read this operating instruction manual thoroughly.

Intended use

This device was developed to transmit audio signals from an audio source to a receiver, using Bluetooth. Any other use is considered unintended use. Unauthorised modification or reconstruction is not permitted. Under no circumstances open the devices or complete any repair work yourself.

For users wearing a pacemaker

It is recommended that if you have a pacemaker fitted you check with a medical expert or your health adviser before using this product. Switch the device off immediately if you notice or suspect any adverse effects.

Power adapter plug

Attention: Only use the power adapter plug contained in the material supplied since other power adapter plugs could damage the transmitter. Ensure access to the power adapter plug is not obstructed by furniture or such.

Pay attention that the plug and cable are in perfect condition. Kinked or worn cable represents the risk of a fatal accident!

The power adapter plug supplied fulfils the ecodesign requirements of the European Union (Directive 2005/32/EC). This means that, both in an operating state and in an idling state, the power consumption is considerably lower compared to power adapter plugs with an older design.

- Energiesparend
- Economisant l'énergie
- A risparmio di energia
- Energy efficient

Disposal

In order to dispose of your device, take it to a collection point provided by your local public waste authorities (e.g. recycling centre). According to laws on the disposal of electronic and electrical devices, owners are obliged to dispose of old **electronic and electrical devices** in a separate waste container. The adjacent symbol indicates that the device must not be disposed of in normal domestic waste!

Batteries represent a hazard to health and the environment!

Never open, damage or swallow batteries or allow them to pollute the environment. They may contain toxic, ecologically hazardous heavy metals. You are legally obliged to dispose of power packs and batteries at the point of sale or in the corresponding containers provided at collection points provided by local public waste authorities. Disposal is free of charge. The adjacent symbols indicate that the batteries must not be disposed of in normal domestic waste and must be brought to collection points provided by local public waste authorities.

Packaging materials must be disposed of according to local regulations.

Helpline

If you need some help and you're in the UK, please first check and see if your problem can be solved using the troubleshooting guide on page 22. If it can't, please call us on 0844 800 6503 (using a UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays.

If you've a claim on your 2-year guarantee, please contact your supplier.

Declaration of conformity

This device fulfils the requirements stipulated in the EU directive: 1999/5 EU directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity. Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website **www.amplicomms.com**

Maintenance

- Clean the housing surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

Technical data

Transmission mode:	Bluetooth
Range:	Up to 10 meters (open area)
Frequency response:	30 Hz – 20 KHz
Max volume	120 dB
Distortion:	< 0.5%
Signal-to-noise ratio:	> 75 dB
Operating voltage (Receiver):	3.7 V, 300 mA Lithium Polymer Battery
Operation time:	With fully charged battery up to 5 hours
Charging time:	For empty battery approx. 3 hours
Weight Receiver	65 g

Guarantee

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life.

The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material or production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOMMS equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

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Audioline GmbH, D-41460 Neuss