# **BTH 1410-NL**

**Bedienungs-  
anleitung (DE)**

**User Guide (GB)**

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**Istruzioni d'uso (IT)**

**Handleiding (NL)**

**Manual de  
instrucciones (ES)**

**Bruksanvisning (SE)**

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**Bruksanvisning (NO)**

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**Übersicht  
At a glance  
Aperçu  
Riepilogo  
Overzicht  
Información general  
Översikt  
Características gerais  
Oversikt  
Yleiskuvaus**

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- 1 Volume control
- 2 LED indicator
- 3 Off-hook/On-hook button
- 4 Microphone button for room monitoring
- 5 Balance left/right (only applicable when connected to an ordinary earphone/headphone)
- 6 Tone clarity button
- 7 Audio line-out socket for earphone
- 8 Jack for charging cable (micro USB)
- 9 On/Off switch

# Getting started

## Getting started

### Box contents

- 1 Receiver with induction neck-loop
- 1 AC adapter with Micro USB for charging the receiver
- 1 User guide

**Important: please make sure you've read the safety information on page 24 before you set up your device.**

### Charging the battery

Charge the battery for about 3 hours before first use. Switch off the receiver (position OFF).

- 1 Attach the micro USB plug (power adapter) to the charging jack on the receiver.
- 2 Attach the power supply to a wall outlet. The LED indicator will flash green whilst the power supply is connected and the unit is charging.

- **A fully charged battery provides up to 5 hours of listening enjoyment.**
- **To conserve battery power, the receiver automatically shuts off if no signal is detected for a period of approx. 5 minutes or when the level is too low.**
- **Standby time: up to 180 hours (when the receiver is connected to a device but not playing any sound)**
- **Auto off: when the receiver is turned on without any connection to Bluetooth devices, it will be turned off after 5 minutes. To turn on the receiver again, you should first slide the power button to the "OFF" position. Then slide it back to "ON" again.**
- **After Auto off: if the power button is not slid to "OFF" the battery can last up to 1.5 months.**

## Introduction

This neckloop receiver is designed to be used especially in conjunction with T Coil hearing aids. Whether you use monaural or binaural hearing aids, if your aids have a T Coil function the sound will be transmitted wirelessly directly from the neckloop to your instruments. This kind of transmission helps cutting out surrounding noises and deliver louder sound, clearer with less interference.

**PLEASE NOTE – the receiver will transmit sounds to hearing aids with T Coil function only.** Some hearing aids have manual activation of the T Coil often known as 'T' setting. In manual hearing aids you will need to press the switch on your hearing aid to 'T' mode in order to use the neckloop. Other hearing aids have automatic 'T' selection whereby your hearing aid will recognise the neckloop magnetic field and switch to the required mode automatically. Not all hearing aids perform in the same way with this neckloop. Sound performance and interference can be affected by the type and quality of your instrument, and especially by its rating of hearing aid compatibility. If you are unsure about the specification or features of your hearing aid contact your audiologist or manufacturer.

## Bluetooth pairing

- 1 Switch on the receiver.
- 2 Press and hold down the MIC button for approx. 5 seconds. The indicator LED will flash green and blue.
- 3 Now run on the transmitter (e.g. on your mobile phone) the necessary steps. If necessary, refer to the relevant user manual for help. The Bluetooth name of the receiver is "amplicomms".
- 4 If the devices are paired successfully, the indicator LED lights blue constantly.

## NFC pairing

If your sending device (e.g. your mobile phone/smartphone) supports NFC (Near Field Communication), attach the receivers area marked with the NFC-Logo () , as near as possible to the relevant area on the sender for a few seconds. It will then activate the Bluetooth function and pair with each other automatically.

# Operation

## Operation

### Hearing damage due to high volumes!

This receiver allows you to set higher volumes than conventional headphones. This can cause hearing damage in persons with healthy hearing and especially in children.

- Before putting the receiver on, set the volume on the receiver to a low value.
- Do not continuously expose yourself to high volumes.

- 1 Switch the ON/OFF switch on the receiver to ON position.
- 2 Make sure the Bluetooth connection is established (indicator LED lights blue)
- 3 The neck-loop induction cable can transmit the audio signal to the hearing aid device for listening.
- 4 Adjust the desired volume.
- 5 Adjust the tone clarity setting if necessary.

To set the tone clarity briefly press the TONE button repeatedly.

- 1 short beep: Treble boost
  - 2 short beeps: Bass boost and treble boost
  - 3 short beeps: Bass boost
  - 1 long beep: Flat
- 6 Adjust the balance (left/right) if necessary.

The balance control can be used to pan to the required stereo channel content. For example if the speech in a stereo audio signal was more prevalent in the left channel then the balance can be tuned to the left to emphasise the signal better.

**Note:** When the battery level is low, the indicator LED sparkles in blue (Bluetooth mode) or in green (MIC mode). You will hear 3 beeps every minute and 1 double-beep in the forth minute. The receiver will switch off automatically in the fifth minute.

## **The MIC button**

The microphone opening at the lower part of the receiver front picks up the sounds of your surroundings. When the microphone is switched on, you will hear your surroundings louder. You can hear for example your door bell, your telephone or your conversational partner even when you are wearing the receiver.

Press the MIC button to switch the microphone on. The power on indicator will turn to green and one short beeps is to be heard. Make sure that the microphone opening is not covered!

You can hear your surroundings and you use the receiver as an assistive listening device which helps you to better hear other people.

Press the MIC button again to switch off the microphone. The power on indicator will return to blue and two short beeps are to be heard.

**Caution:** When switching between the microphone and music source please be aware that the volume level can change substantially dependant upon signal input levels. As a precaution the volume level should be lowered prior to switching between the microphone and music.

## **The PHONE button**

If you have your receiver connected to a mobile phone / smartphone, you can answer or end a phone call by briefly pressing the phone button. When an incoming call is received you will hear the ringing tone from the mobile phone transmitted to the neck-loop which in turn will inductively couple the signal to the hearing aid. Press and release the 'Phone' button on the BTH1410-NL and then you can converse with the caller. The microphone in the Neck-loop will pick up your voice. To end the call simply press and release the 'Phone' button again.

## **Opening/closing the neckloop**

It is possible to open and close the neckloop to the receiver base for easier wearing and removing. This is achieved by gently turning the right part of the neckloop anti-clockwise 1/8 turn and then pulling the plug out. Re-assembly is the reverse process.

## **Troubleshooting**

### **No sound**

- Ensure that the receiver power ON/OFF switch is switched to ON.
- The charge level of the battery inside receiver may be too low. Recharge the battery for approx. 3 hours.
- Ensure that the Bluetooth connection is established.
- The receiver output level is set too low. Adjust the volume to a suitable level.

### **Distortion**

- The charge level of the battery inside the receiver may be too low. Recharge the battery for approx. 3 hours.
- The receiver may be too far away from the Bluetooth transmitter (e.g. smartphone etc.). Move closer to it.

# Appendix

## Appendix

### Safety Information

- Please read this operating instruction manual thoroughly.
- **For safety reasons do not use this product near machinery as neckloop can become tangled in moving parts.**

### Intended use

This device was developed to transmit audio signals from an audio source to a receiver, using Bluetooth. Any other use is considered unintended use. Unauthorised modification or reconstruction is not permitted. Under no circumstances open the devices or complete any repair work yourself.

### For users wearing a pacemaker

It is recommended that if you have a pacemaker fitted you check with a medical expert or your health adviser before using this product. Switch the device off immediately if you notice or suspect any adverse effects.

### Power adapter plug

**Attention: Only use the power adapter plug contained in the material supplied since other power adapter plugs could damage the transmitter. Ensure access to the power adapter plug is not obstructed by furniture or such.**

Pay attention that the plug and cable are in perfect condition. Kinked or worn cable represents the risk of a fatal accident!

The power adapter plug supplied fulfils the ecodesign requirements of the European Union (Directive 2005/32/EC). This means that, both in an operating state and in an idling state, the power consumption is considerably lower compared to power adapter plugs with an older design.

- Energiesparend
- Economisant l'énergie
- A risparmio di energia
- Energy efficient

## Disposal

In order to dispose of your device, take it to a collection point provided by your local public waste authorities (e.g. recycling centre). According to laws on the disposal of electronic and electrical devices, owners are obliged to dispose of old **electronic and electrical devices** in a separate waste container. The adjacent symbol indicates that the device must not be disposed of in normal domestic waste!

### **Batteries represent a hazard to health and the environment!**

Never open, damage or swallow batteries or allow them to pollute the environment. They may contain toxic, ecologically hazardous heavy metals. You are legally obliged to dispose of power packs and batteries at the point of sale or in the corresponding containers provided at collection points provided by local public waste authorities. Disposal is free of charge. The adjacent symbols indicate that the batteries must not be disposed of in normal domestic waste and must be brought to collection points provided by local public waste authorities.

**Packaging materials** must be disposed of according to local regulations.

## Helpline

If you need some help and you're in the UK, please first check and see if your problem can be solved using the troubleshooting guide on page 22. If it can't, please call us on 0844 800 6503 (using a UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays.

If you've a claim on your 2-year guarantee, please contact your supplier.

## Declaration of conformity

This device fulfils the requirements stipulated in the EU directive:

1999/5 EU directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity.

Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website **[www.amplicomms.com](http://www.amplicomms.com)**

## Maintenance

- Clean the housing surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

## Technical data

Transmission mode:	Bluetooth
Range:	Up to 10 meters (open area)
Frequency response:	30 Hz – 20 KHz
Distortion:	< 0.5%
Signal-to-noise ratio:	> 75 dB
Operating voltage (Receiver):	3.7 V, 300 mA Lithium Polymer Battery
Operation time:	With fully charged battery up to 5 hours
Charging time:	For empty battery approx. 3 hours
Weight Receiver	65 g

## Guarantee

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life.

The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material or production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOMMS equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

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Audioline GmbH, D-41460 Neuss