RingFlash 100/200

Bedienungsanleitung (DE)
Mode d'emploi (FR)
Operating Instructions (UK)
Gebruiksaanwijzing (NL)
Manual de instrucciones (ES)

Istruzioni per l'uso (IT)
Bruksanvisning (NO)
Bruksanvisning (SE)
Käyttöohje (FI)
Manual de instruções (PT)

RingFlash 100/200

RingFlash 100/200

Bedienungsanleitung (DE)	3
Mode d'emploi (FR)	15
Operating Instructions (UK)	27
Gebruiksaanwijzing (NL)	39
Manual de instrucciones (ES)	51
Istruzioni per l'uso (IT)	63
Bruksanvisning (NO)	75
Bruksanvisning (SE)	87
Käyttöohje (FI)	99
Manual de instruções (PT)	111

Sicherheitsinformationen

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Safety Information

Please read this operating instruction manual thoroughly.

Under no circumstances open the device or complete any repair work yourself.

Installation location

Prevent excessive exposure to smoke, dust, vibration, chemicals, moisture, heat and direct sunlight.

Power supply

Caution: Only use the power adapter plug supplied because other power supplies could damage the product. Ensure access to the power adapter plug is not obstructed by furniture or such.

Disposal

In order to dispose of your device, take it to a collection point provided by your local public waste authorities (e.g. recycling centre). According to laws on the disposal of **electronic and electrical devices**, owners are obliged to dispose of old electronic and electrical devices in a separate waste container.

The adjacent symbol indicates that the device must not be disposed of in normal domestic waste!

Batteries represent a hazard to both health and the environment!

Never open, damage or swallow batteries, nor allow them to get into the environment.

Safety information

They may contain toxic and ecologically hazardous heavy metals. You are legally obliged to dispose of batteries at the point of sale or at appropriate collection points provided by your local public waste authorities.

Disposal is free of charge. The adjacent symbols indicate that batteries must not be disposed of in normal domestic waste and but at authorised collection points.

Packaging materials must be disposed of according to local regulations.


Starting Up

Check the contents of the package

Contents	RF100	RF200
RingFlash unit	1	1
Power adaptor plug	1	1
Telephone connection cable	1	1
Doorbell unit	0	1
12 Volt battery	0	1
Fixing screws and wall plugs	0	4
Operating Manual	1	1

Connecting the base

Connect the base as illustrated in the diagram. For reasons of safety, only use the power adaptor plug and telephone connection cable supplied.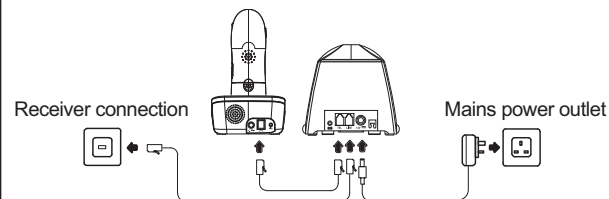

As soon as you switch the mains power on, the RingFlash unit will beep to confirm that it has been installed correctly. If you do not hear a beep, please unplug the power connection, wait for 1 minute and then plug in the power again.


Starting Up

 In some cases where the amplicomms Ring Flash is used in conjunction with older phones or those phones that do not comply to European Standard ES 203021 connectivity, either the RingFlash or telephone may not function correctly. If this happens you may notice the Ring Flash does not flash or ring or it is not possible to hear dial tone in the receiver when the telephone handset is lifted. To resolve this please retain your existing telephone line cord with your phone and use a double adaptor in the telephone socket (not supplied). Plug the telephone into one side of the double adaptor then plug your Ring Flash into the other available socket on the adaptor.

Connecting the doorbell unit (RingFlash 200 only)


Remove the 4 screws from the back of the Doorbell.

The Doorbell has a default setting of Bell 1, if you wish to change this to Bell 2, please move the switches inside the Doorbell to the positions indicated below.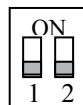

Bell 1


The stop indicator button on the RingFlash unit will be red when the Doorbell is pressed.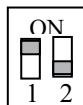

Bell 2

The stop indicator button on the RingFlash unit will be green when the Doorbell is pressed.

Insert the 12 Volt battery supplied, taking note to position the battery the correct way around. Replace the back cover, position and tighten the 4 screws.

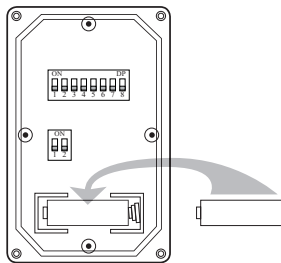👉 You only need to change the Bell setting if you have more than one Doorbell connected to your RingFlash unit. The two options are provided so that you will know from the colour of the light on the STOP button which Doorbell has been pressed and which Door to answer.

Unclip the front cover of the Doorbell by placing a small coin into the slot at either end of the doorbell and twisting. Using the 4 screws and wall plugs provided, attach the Doorbell outside your front door.

Shaker

You can plug a shaker accessory (not included) into the socket '((o))' on the back of the RingFlash unit. The shaker will then vibrate when you receive an incoming call.

👉 For details of where to purchase a compatible Shaker, please go to www.amplicomms.com.

Your RingFlash 100/200

STOP button


Front view

Telephone to
RingFlash socket

Shaker socket

Telephone line
connection to
wall socket


Power adaptor
socket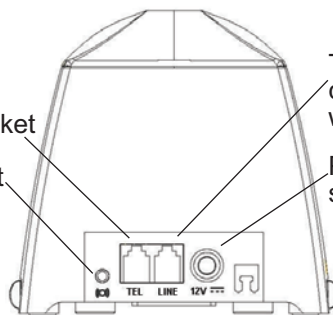

Rear view

Ring tone
selection


Doorbell tone
selection
(RingFlash 200 only)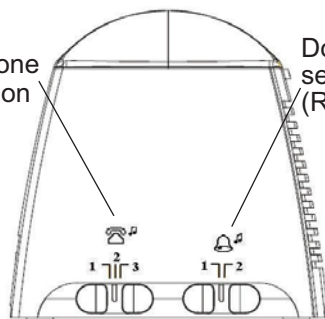

Left side

Volume control

Alarm mode

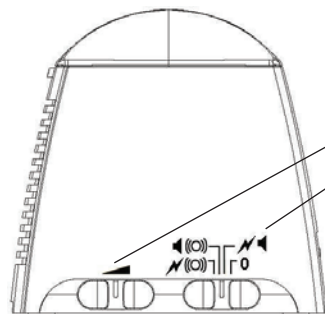Right side

Using your RingFlash 100/200

Set the ring tone

Your RingFlash unit has 3 ring tones to choose from. Just move the slider switch on the left hand side of your product to select ring tone 1, 2 or 3.

Set the Doorbell tone (RingFlash 200 only)

You can choose from two different Doorbell tones, just move the slider on the left hand side of your product to select Doorbell tone 1 or 2.

Set the alarm mode

There are 4 alarm modes to choose from. Please use the slider on the right hand side of the product to select one of the following alarm modes.

Mode	Ring alarm	Bell alarm
0	Off	Bell sound from the speaker
	Strobe light and speaker	Bell sound from speaker and strobe light
	Speaker and external shaker	Bell sound from speaker and external shaker
	Strobe light and external shaker	Bell sound from speaker, strobe light and external shaker

Stop the ring alarm

Press the STOP button on the front of the unit to stop the ring alarm for that call, even if the telephone line continues to ring. The ring alarm will return to its original setting for the next call.

Using your RingFlash 100/200

Using the Doorbell (RingFlash 200 only)

Press the Doorbell button, the red light in the button will turn on. The RingFlash unit will sound the Doorbell tone that you have set (tone 1 or tone 2).

Press the STOP button on the RingFlash unit to stop the Doorbell tone. The red light on the Doorbell button will remain on for 1 minute and then turn off.


Compatibility with other local devices (RingFlash 200 only)

When using the RingFlash 200 and Doorbell there may occasionally be a compatibility problem with other local devices. You can change the code underneath the RingFlash unit and inside the doorbell to solve this problem.

Underneath the RingFlash unit you will find 8 little switches underneath a cover. You can change the setting of these switches to overcome any local compatibility issues.

When you have changed the setting of the 8 switches on the RingFlash unit you will need to change the 8 switches inside the Doorbell to match.

Unscrew the Doorbell from outside the Front Door. Remove the 4 screws from the back of the Doorbell. Inside you will see 8 little switches, change the setting of these switches to match those on the RingFlash unit.

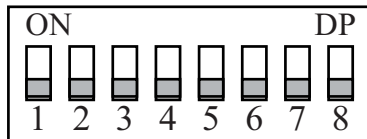Replace the back cover, position and tighten the 4 screws. Refix the Doorbell outside your front door as before.

Wallmounting

Using appropriate screws and wall plugs (not supplied), you can wallmount your RingFlash unit using the keyholes on the base.

Maintenance/Guarantee

Maintenance/Guarantee

Maintenance

- Clean the product surfaces with a soft, fluff-free cloth
- Never use cleaning agents or solvents

Guarantee

Amplicomms equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble-free functioning and a long service life. The terms of the guarantee do not apply where the cause of the equipment malfunction is the fault of the telephone network operator or any interposed private branch extension system. The terms of guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material of production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee. In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return it to the sales outlet in which you

purchased the Amplicomms equipment together with the purchase receipt. All rights to claims under the terms of the guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase and transfer of our products, claims under the terms of guarantee can no longer be asserted.

Declaration of Conformity

This device fulfils the requirements stipulated in the EU directive:

1999/5 EU directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity.

Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website www.amplicomms.com.

Audioline GmbH, D-41460 Neuss

Important Notes on the Guarantee

Important Notes on the Guarantee

The period of guarantee for the Amplicomms product begins on the date of purchase. Rights to claims under the terms of guarantee relate to deficiencies resulting from material or production faults. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, normal wear and tear, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

CLAIMS UNDER THE TERMS OF GUARANTEE MAY ONLY BE ASSERTED TO THE RETAILER WHERE THE EQUIPMENT WAS PURCHASED, WHEREBY THE PURCHASE RECEIPT MUST ALSO BE SUBMITTED.

TO PREVENT UNNECESSARY LONG DELAYS, PLEASE DO NOT SEND EQUIPMENT UNDER COMPLAINT TO OUR OFFICE ADDRESS.

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AUDIOLINE GmbH