# PTV110

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### Sicherheitsinformationen

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## Operating Instructions

### Safety Information

Please read this operating instruction manual thoroughly. Under no circumstances open the device or complete any repair work yourself.

### Disposal

You are obliged to dispose of consumable goods properly in accordance with the applicable legal regulations. The adjacent symbol on the product indicates that electrical and electronic apparatus no longer required must be disposed of separately from domestic waste.

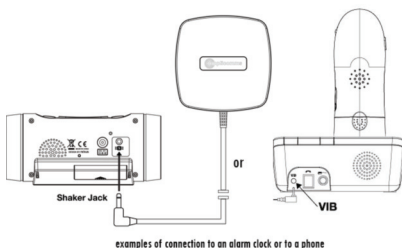
**Electrical and electronic devices** must be disposed of at suitable collection points provided by the public waste authorities.

**Packaging materials** must be disposed of according to local regulations.

### Using the PTV 110

When the 'Shaker' feature on your compatible amplicomms product is set to 'On' the PTV 110 will vibrate each time you receive an incoming call or in case of an alarm.

- 1 Connect the product as illustrated in the diagram.
- 2 Set the 'Shaker' feature on your amplicomms product to 'On' (refer to your amplicomms operating instructions).
- 3 Place the PTV 110 under your pillow/or cushion etc. to alert you to an incoming call or in case of an alarm.

### Maintenance

- Clean the product surfaces with a soft, fluff-free cloth
- Never use cleaning agents or solvents

### Guarantee

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble-free functioning and a long service life. The terms of the guarantee do not apply where the cause of the equipment malfunction is the fault of the telephone network operator or any interposed private branch extension system. The terms of guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material of production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee. In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device.

Replacement parts or devices become our property. Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return it to the sales outlet in which you purchased the AMPLICOMMS equipment together with the purchase receipt. All rights to claims under the terms of the guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet. Two years after the purchase and transfer of our products, claims under the terms of guarantee can no longer be asserted.

### Declaration of Conformity

This device fulfils the requirements stipulated in the EU directive: 2014/30/EU relating to electromagnetic compatibility. Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website [www.amplicomms.com](http://www.amplicomms.com).

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Utdelat by: Audioline GmbH, D-41460 Neuss